

DIRECTIONS FOR USE

Allow the test device, and urine specimen to come to room temperature [15-30°C (59-86°F)] prior to testing.

- 1) Remove the test device from the foil pouch.
- 2) Remove the cap from the test device. Label the device with patient or control identifications.
- 3) Immerse the absorbent tip into the urine sample for 10-15 seconds. Urine sample should not touch the plastic device.
- 4) Replace the cap over the absorbent tip and lay the device flatly on a non-absorptive clean surface.
- 5) Read results at 5 minutes.

DO NOT INTERPRET RESULT AFTER 5 MINUTES.

