

SalivaConfirm Premium with Prescription Drugs

11 PANEL SALIVA DRUG + ALCOHOL TEST



A Revolutionary One Step Oral Fluid Drug Test

In drug analysis, oral fluid drug screening has certain advantages over the screening of other biological fluids such as urine and blood. Oral fluid is readily accessible, and is less likely to be adulterated. With oral fluid tests the drugs may be detected immediately after ingestion, even before they are metabolized and would show up in urine.

SalivaConfirm-Premium is designed to work at a lower detection level for all test drugs than those detected in urine samples. SalivaConfirm premium oral screening for drugs of abuse detects the presence of parent compounds and drug metabolites.

SalivaConfirm™- Premium Saliva Test:

- **Unique Product:** The only saliva test kit in the market that can test for 11 drugs plus alcohol
- **Easy and convenient**
- **Accurate & Reliable**
- **Easy to Use and Convenient:** Easy to administer anywhere and anytime. No need for special training or expensive equipment to administer the test
- **No more controlled collection sites or fees**

Sensitivity

Amphetamine (AMP)	50 ng/mL
Methamphetamine (mAMP)	50 ng/mL
Cocaine (COC)	20 ng/mL
Opiates (OPI)	40 ng/mL
Marijuana (THC)	12 ng/mL
Phencyclidine (PCP)	10 ng/mL
Benzodiazepines (BZO)	50 ng/mL
Oxycodone(OXY)	50 ng/mL
Methadone(MTD)	75 ng/mL
Barbiturates (BAR)	300 ng/mL
Buprenorphine (BUP)	10 ng/mL
Alcohol(ACL)	> 0.02 % B.A.C

* For Forensic use only , not intended for home use.

www.confirmbiosciences.com


Email: info@confirmbiosciences.com

6310 Nancy Ridge Drive #103
San Diego CA 92121
Tel: 888- 526-6347



Instructions


Allow the test device to reach room temperature [15-30°C (59-86°F)] prior to testing. Do not place anything in the mouth including food, drink, gum, or tobacco products for at least 10 minutes prior to collection of oral fluid specimen.

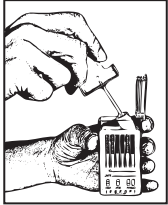


STEP 1: Remove the collection stick and test tube from the sealed pouch.




STEP 2: Insert the sponge end of the collection stick into mouth and soak sponge into saliva for 3 minutes. Note: Time should be longer for people of little saliva. If the amount of saliva pressed into the test tube is not adequate for testing, collect more with another new collection stick and express the saliva into tube again

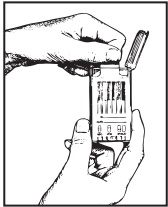

STEP 3: Hold the test tube vertically and place the collection stick with saturated sponge into the test tube. Make sure to fit the groove of collection stick onto the guide rail of test tube and press the collection stick to full extent.

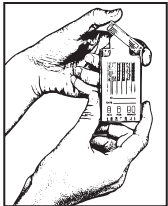

STEP 4: Press down the lid to close the test tube. Keep the test tube vertically until you begin to read the test results.

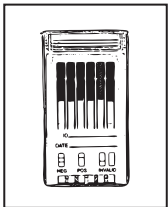

STEP 5: Read results of alcohol test at 2 minutes and drug tests at 10 minutes. (If there is a label over reading window, peel off the label to read test results.) Do not read alcohol test result after 5 minutes and drug test results after 1 hour.

Result Indicators

Allow the test device to reach room temperature [15-30°C (59-86°F)] prior to testing. Do not place anything in the mouth including food, drink, gum, or tobacco products for at least 10 minutes prior to collection of oral fluid specimen.



Negative
(2 Lines)




Positive
(1 Line)



Invalid
No Line(s) or
no Control Line(s)

Alcohol Test Results

Alcohol Presumptive Positive Result: The Alcohol test produces a color change to green to blue in the presence of salivary alcohol 0.02 % B.A.C. or higher. At higher alcohol concentration near 0.30% B.A.C., the color may change to a dark blue-gray.

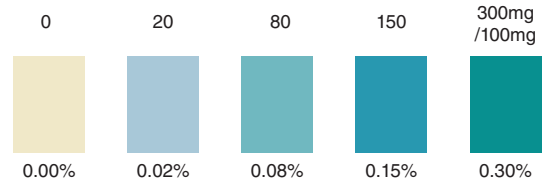

SalivaConfirm Premium
with Prescription Drugs